




HOT & COLD GEL PACK

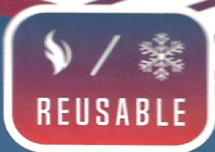

- LONG LASTING TECHNOLOGY
- HEAT & COOL THERAPY
- ELASTIC SECURE STRAPPING
- MULTI-PURPOSE (BACK, STOMACH,
SHOULDER & MENSTRUAL PAIN)



DO NOT OPEN, PUNCTURE OR STRIKE THE PACK

COOLING INSTRUCTIONS:

Place product in the freezer for a minimum of 2 hours or in a refrigerator until the desired cool temperature is produced. Check temperature is suitable prior to use.

HEATING INSTRUCTIONS:

Place product in microwave. Check product is resting flat. Microwave for 30 seconds. Make sure the product temperature is suitable before application. If more heat is needed, re-heat in microwave for a 10 second period and check again. Before re-heating, thoroughly check all areas of product as some areas may be a higher temperature (frequently occurs at the product edges). Repeat process until fully satisfied.

PRECAUTION

External use only. DO NOT use product if overheated DO NOT place product directly on to skin. DO NOT tamper with the product. If the product pack breaks and the gel fluid contacts skin or eyes immediately rinse with water and and seek medical attention. If ingested immediately wash mouth and seek medical attention. To be used under adult supervision only.

INGREDIENTS

Sodium polyacrylate, water, glycerin.

MADE IN CHINA

A Clifton's® Product

Clifton Supplies Ltd

Registered no:12396290

Hillsdown House, 32 Hampstead High St,
London, NW3 1QD

