

WiFi Otoscope



Customer Service: looya@188.com

Download APP

iOS

Search for “Time siso” on APP Store to download and install.

Android (international)

Search for “Time siso” on Google Play to download and install.

Noted: After installation, please allow the mobile phone to use the APP.



Or scan the QR code to download “Time siso”.



Turn on /off



Cap — Turn on: Take off the cap.
 — Turn off: Close the cap.




Charging: Type-C port USB cable.

Easy to charge with PC, Laptop, power bank, or USB plug.

Connection with "Time siso" APP


1

1. Turn on the otoscope and connect Wifi "Timesiso-XXXX", the password is 12345678.

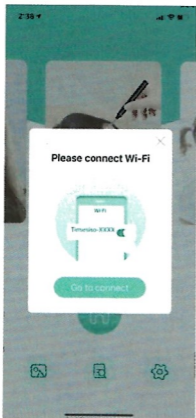

2

2. Open the "Time siso" APP, click the "home" sign to enter ear camera.

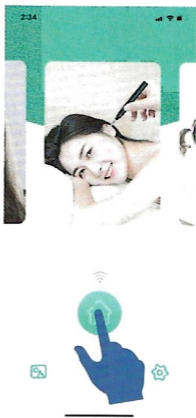

Please noted: Only 1 device can connect WiFi otoscope when using, if you change to another device, need to disconnect the wifi and restart.

Assemble the Ear Spoons

Kid' s ear spoon

Adult' s ear spoon

360°ear spoon

Adult' s ear cover

Kid' s ear cover

Noted: Please slowly extend the ear spoon into your ears, not too deep, to avoid hurting your ears.

Pakete Inhalt

1 x WiFi otoscope
6 x Ear spoons
2 x Ear cover
2 x Alcohol swab
1 x USB cable
1 x User manual

Produkt Parameter

Camera diameter: 3.9 mm

Image sensor: CMOS

Resolution: Native 720P

Viewing angle: 70°

LED lights: 6pcs

Focal length: 1.4~2 cm

Input: 5V 1A

Battery capacity: 220mAh

Charging time: about 1.5 hours

Working time: 60 minutes

Support system: Android 4.2 and above / iOS 4.2 and above

Fehlerbehebung

Q: When taking off the cap, LED doesn't light up and the camera no response

A: Please charge the otoscope

Q: APP crashed

A: Please uninstall the app and scan the QR code to reinstall the app

Q: LED light flash red

A: Please charge the otoscope

Q: Image fails to display

A: Please restart the otoscope and APP, connect to WiFi

Q: WiFi connection break or cannot connect to WiFi

A: Please restart the otoscope and APP, reconnect to WiFi

Postfach nach dem Verkauf

Dear customer,

Thanks so much for choosing LOOYA.

If you encounter any problems, please contact us firstly.

After-sales service mailbox: **looya@188.com**

We will help you solve problems at once.

Thank you!